



# Procedure Card

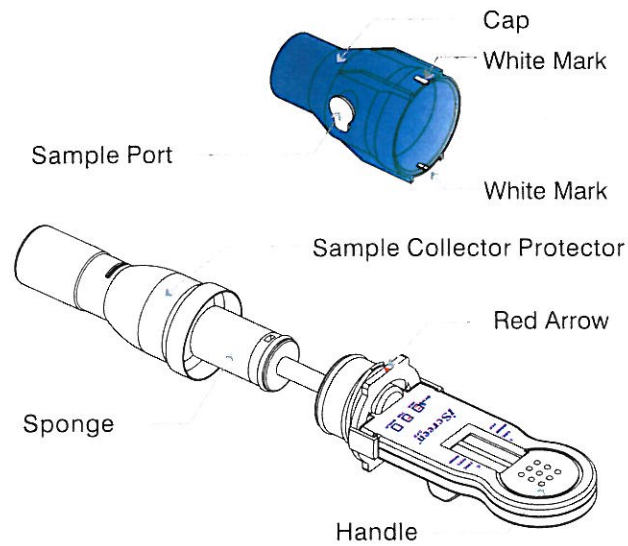


## Section I: Sample Collection

- 1 • Instruct the donor not to place anything in the mouth including food, drink, gum or tobacco products for at least 10 minutes prior to collection.
- 2 • Remove Sample Collector Protector.

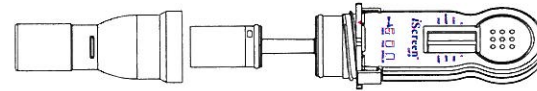


- 3 • Insert Sponge end of collector into mouth.
- Actively swab the inside of the mouth and tongue with Sponge. As soon as the Sponge softens slightly, gently press the Sponge between the tongue and teeth to ensure **complete saturation**. The Sponge is saturated when no hard spots can be felt. Collect for at least three (3) minutes before removing Sponge.
- Hand the saturated collector over to the tester.

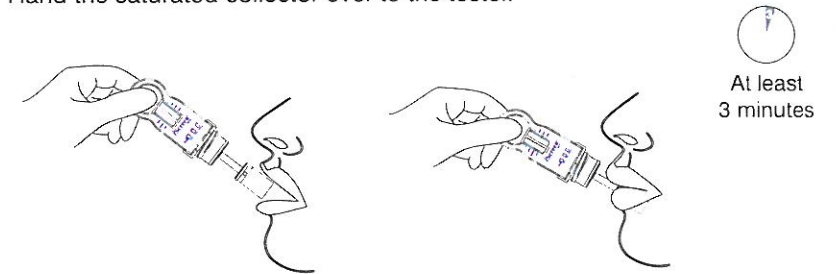


## Section II: Testing

- 1 A. Align the Red Arrow on the device with either one of the **White Marks** on the Cap. Insert the collector **vertically** into the Cap and **press down firmly until the Cap reaches the thread**.
- B. Twist the Handle clockwise 180° to tighten the Cap until the **Red Arrow** lines up with the other **White Mark**.
- C. Place the test device horizontally on a clean and level surface with the result window facing up.

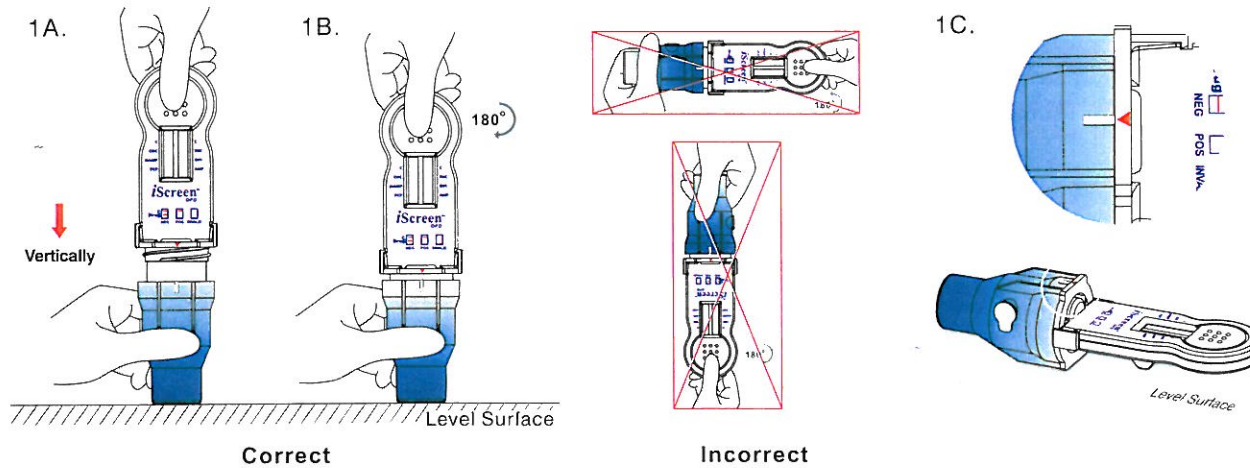

- 2 • Read results at 10 minutes. Do not read results after 1 hour.

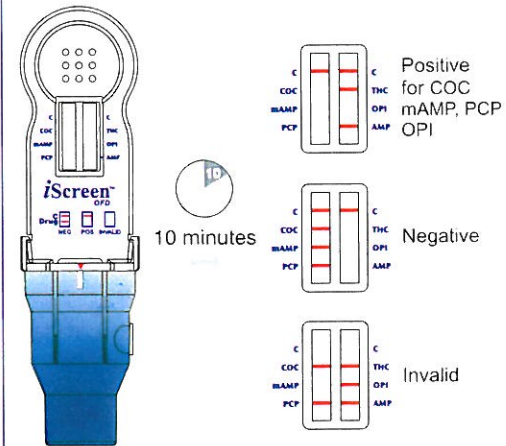

- All positive results are presumptive and should be confirmed by an alternate method. The preferred confirmatory method is GC/MS or GC/MS/MS.

- 3 • If results are positive, secure Cap with Security Seal and send to laboratory for confirmation.

